

ANSWER²


Children & Adolescent Sizing Chart

Important: The ANSWER 2 shoes arrive with removable inserts that increase your fitting capabilities, be certain to remove the inserts before attempting to fit the shoe to the brace side.

To check that your sizing chart printed accurately, the big rectangle box should measure 4" wide x 10" high.

Length Measurement

- 1 Place heel on chart**
 - Align heel of foot/AFO/KAFO with the bottom line.
 - If foot is longer than brace, include foot
- 2 Trace foot/AFO/KAFO**
 - Match longest point to number on right
 - If you are between or slightly above line, round up
- 3 Necessary Adjustments for Circumference**
 - If you are between or slightly above line, round up:
Increase 1/2 full size
 - Articulating hinges, posterior stop and instep strapping:
Increase 1 full size
 - Toe strapping, the additions above will change the fit of the shoe:
Increase 1/2 full sizes

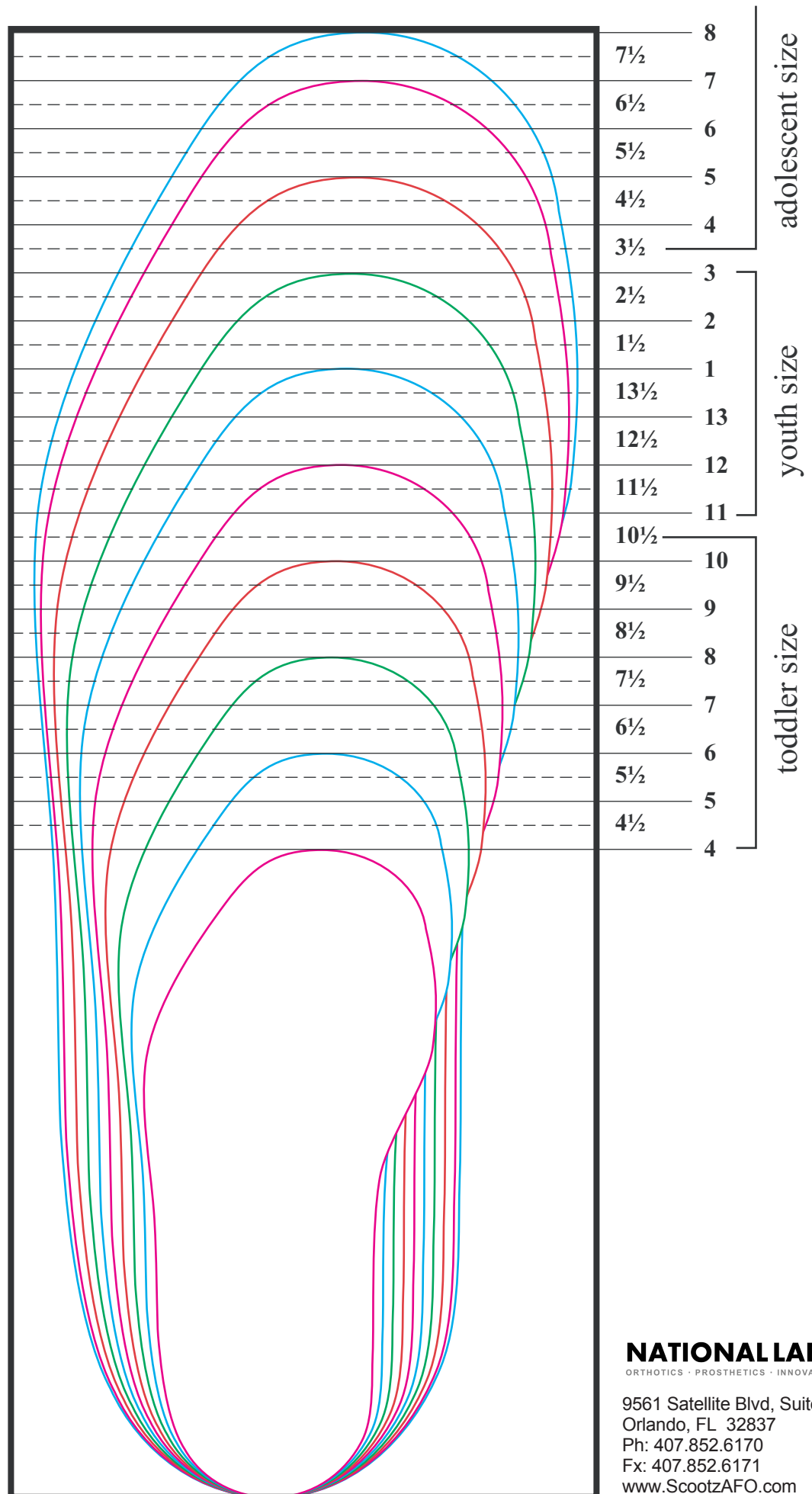

NATIONAL LABS
ORTHOTICS · PROSTHETICS · INNOVATION

9561 Satellite Blvd, Suite 350
Orlando, FL 32837
Ph: 407.852.6170
Fx: 407.852.6171
www.ScootzAFO.com